

MODEL	SMART TENS	
<p style="text-align: center;"><b>CONTENTS:</b></p> <ul style="list-style-type: none"> <li>• Electrotherapy TENS/EMS with graphic display</li> <li>• Belt clip</li> <li>• 2 connection cables</li> <li>• 4 pre-gelled electrodes</li> <li>• User Manual</li> <li>• 4 batteries AAA Size</li> <li>• Carriage bag</li> </ul>		
<p style="text-align: center;"><b>MAIN TECHNICAL FEATURES:</b></p> <ul style="list-style-type: none"> <li>• Combined electrotherapy unit with 2 independent channels;</li> <li>• compensated biphasic square wave: current quantity from positive pole to negative one is constant to avoid hazardous thermal effect of polarization;</li> <li>• 6 TENS stored programs;</li> <li>• 1 TENS free program;</li> <li>• 5 EMS stored programs;</li> <li>• 1 EMS free program;</li> <li>• 2-120 Hz waveform frequency;</li> <li>• 50-400µs width impulse;</li> <li>• 60 mA maximum intensity;</li> <li>• easy to use digital keyboard;</li> <li>• on-line guide on display for fast instructions;</li> <li>• operating with 4 batteries AAA Size;</li> <li>• CE0476 medical device.</li> </ul>		
<p style="text-align: center;"><b>TENS and EMS</b></p> <p>TENS represents the therapeutic technique mainly used for analgesic purposes in physiotherapy. It finds application in a wide variety of pain problems, as acute or chronic pains, muscular pains, post – operated pains and also pains caused by pregnancy. It's usually divided according to its action mechanism in tens conventional, tens endorphinic, tens sequential, tens maximum value, tens Burst.</p> <p>EMS is a stimulating technique that promotes muscular contractions thanks to electrical impulses. As happening in TENS stimulation impulses are applied to patient thanks to use of electrodes collocated next to the muscles that have to be treated. In physiotherapy EMS is used for muscular rehabilitation post – traumatized and post – operated, and also as a complementary aid in sport activity with both therapeutic and sport or beauty purposes.</p> <p>IACER has combined in one product some basic programs used in TENS treatments together with EMS classic protocols.</p> <p>SMART TENS is the only product in its category able to customize 1 TENS treatment and 1 EMS treatment in according to user's needs or physiotherapist's requirements. SMART TENS offers therefore a wider possibility to treat acute and chronic pains (TENS) and muscular strengthening and stimulation (EMS).</p>		
<p><b>Dimensions:</b> 68 X 110 X 31 mm  <b>Weight:</b> 85 gr (batteries excluded)</p>		